


REMOTE OUTPUTS:
 SWITCHED NEGATIVE OUTPUTS-
 500mA EACH. TURN ON WHEN ZONE
 IS LATCHED.

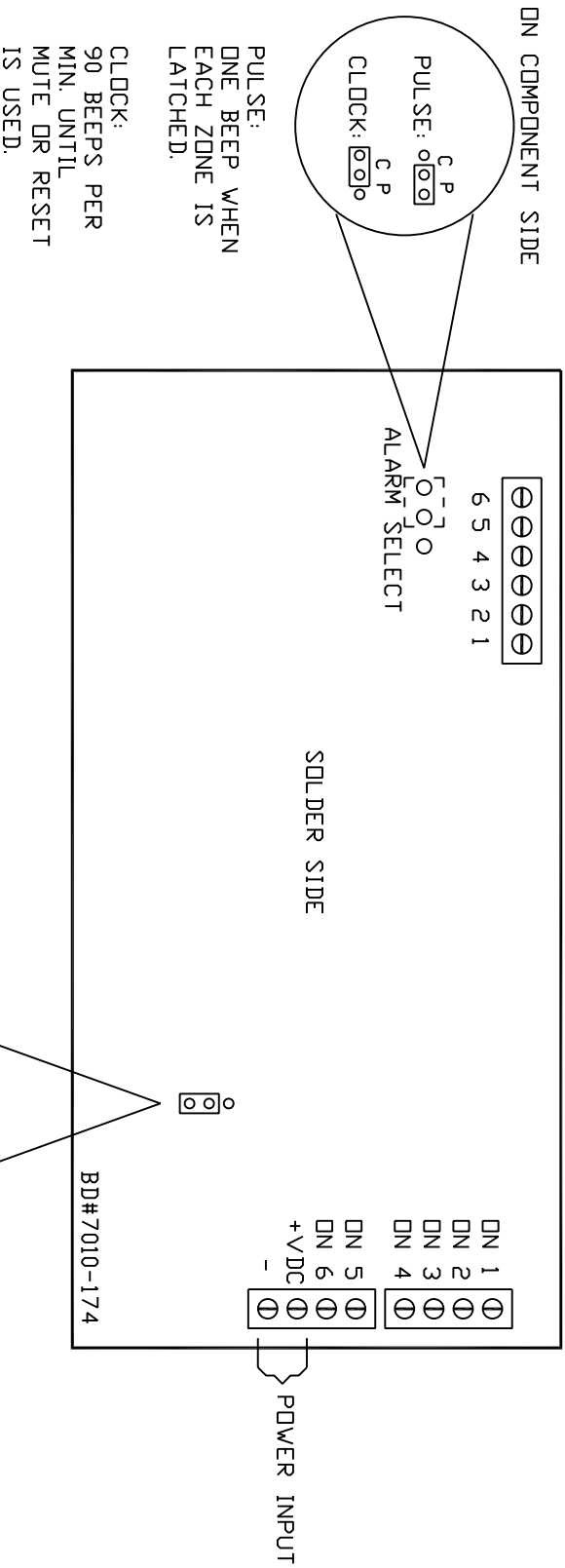

A MOMENTARY CONNECTION OF POWER SUPPLY NEGATIVE TO AN "DN" TERMINAL WILL LATCH ON THAT ZONE AND BEGIN THE ALARM. THE RESET SWITCH WILL SILENCE THE ALARM AND CANCEL ALL ZONES. THE MUTE SWITCH WILL SILENCE THE ALARM WITHOUT RESETTING THE LATCHES.

NOTE: EACH TIME A NEW ZONE IS LATCHED THE ALARM WILL RESTART.



CORNELL Communications, Inc.
 sales@cornell.com 414-351-4660
 Web: www.cornell.com 414-351-4657(fax)

DATE: SEPTEMBER 22, 2006
 TITLE: SINGLE STATUS SYSTEM 4006L

SYSTEM: 4000 NURSE CALL
 DRAWING # 90037-06 DA4006LW